


TRI-NEX 06.0mm Tapered Implants - Drill Sequence

Tapered (IA-LH-60)


Soft Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm
Ø6.0mm

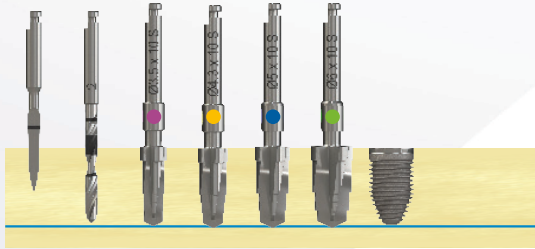


Medium Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm
Ø6.0mm

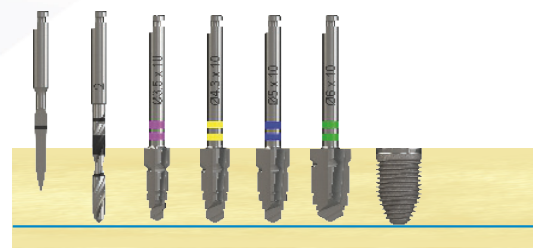

Dense Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm
Ø6.0mm
D-TAP-L60

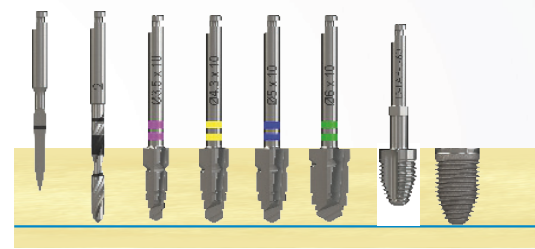

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.
(illustration is for 10mm implants.)



SOUTHERNIMPLANTS

Innovative Treatment Solutions

www.southernimplants.com