


## Parallel Walled (BA)


### Soft Bone

D-3Spade  
Ø2.0mm  
Ø3.07mm  
Ø4.0mm

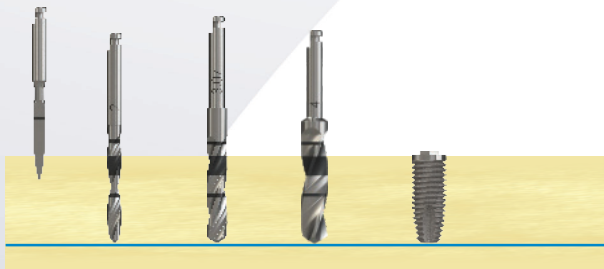


### Medium Bone

D-3Spade  
Ø2.0mm  
Ø3.07mm  
Ø4.3mm  
D-CSS-5

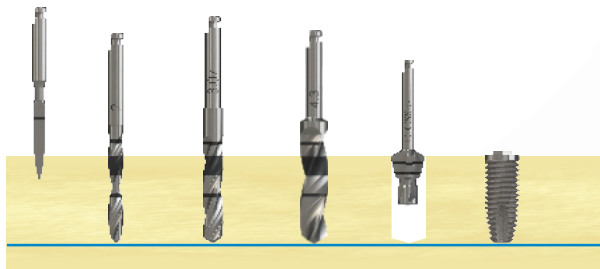

### Dense Bone

D-3Spade  
Ø2.0mm  
Ø3.07mm  
Ø4.6mm  
D-CSS-5  
D-TAP-BA

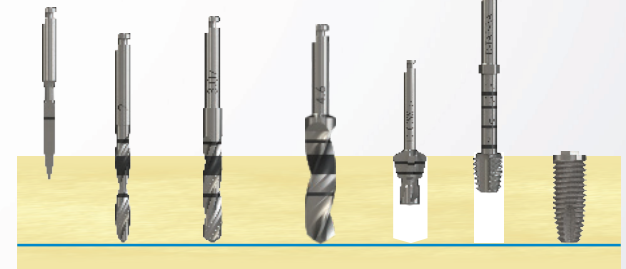


NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.  
(Illustration is for 13mm implants)

## Tapered (BAT / MSc-BAT)


### Soft Bone

D-3Spade  
Ø2.0mm  
Ø3.3mm  
Ø4.0mm

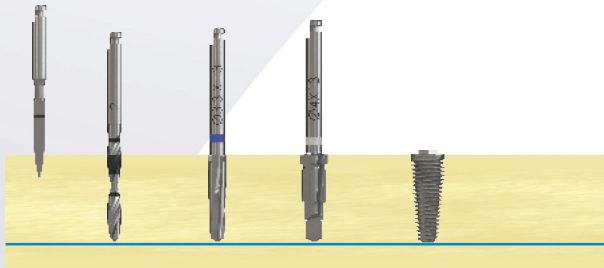


### Medium Bone

D-3Spade  
Ø2.0mm  
Ø3.3mm  
Ø4.0mm  
Ø5.0mm

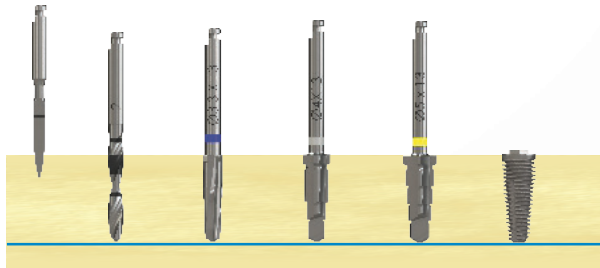

### Dense Bone

D-3Spade  
Ø2.0mm  
Ø3.3mm  
Ø4.0mm  
Ø5.0mm  
D-TAP-BAT

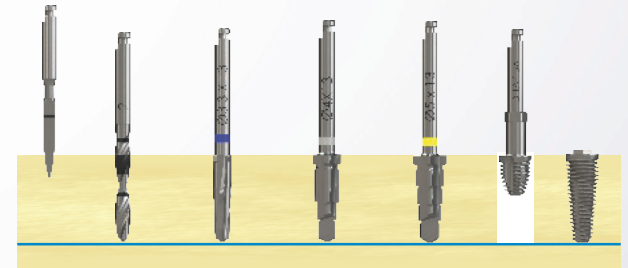

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.  
(Illustration is for 13mm implants)