


External Hex 07.0mm MAX Implants (Tapered) - Drill Sequence

Tapered (MAX-7 / MSc-MAX-7)


Soft Bone

D-3Spade
Ø2.0mm
Ø3.3mm
Ø4.0mm
Ø5.0mm
Ø6.0mm

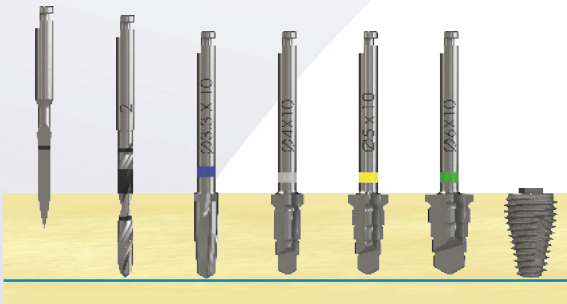


Medium Bone

D-3Spade
Ø2.0mm
Ø3.3mm
Ø4.0mm
Ø5.0mm
Ø6.0mm
Ø7.0mm

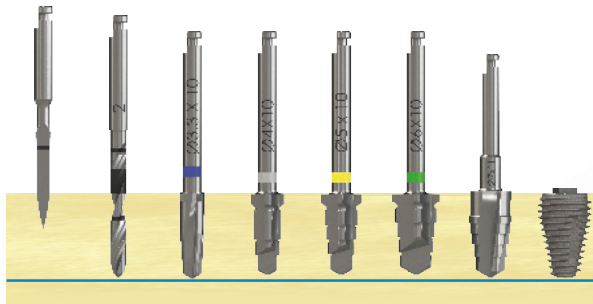

Dense Bone

D-3Spade
Ø2.0mm
Ø3.3mm
Ø4.0mm
Ø5.0mm
Ø6.0mm
Ø7.0mm
D-TAP-MAX7-11

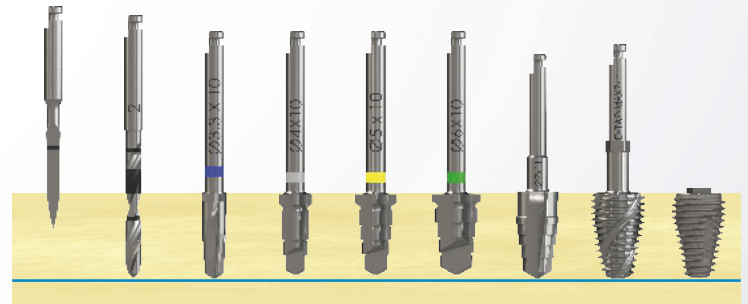

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.
(Illustration is for 13mm implants)



SOUTHERNIMPLANTS

Innovative Treatment Solutions

www.southernimplants.com