


External Hex 08.0mm MAX Implants (Tapered) - Drill Sequence

Tapered (MAX-8 / MSc-MAX-8)


Soft Bone

D-3Spade
Ø2.0mm
Ø3.3mm
Ø4.0mm
Ø5.0mm
Ø6.0mm
Ø7.0mm

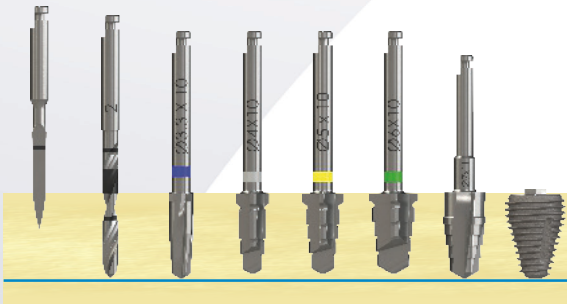


Medium Bone

D-3Spade
Ø2.0mm
Ø3.3mm
Ø4.0mm
Ø5.0mm
Ø6.0mm
Ø7.0mm
Ø8.0mm

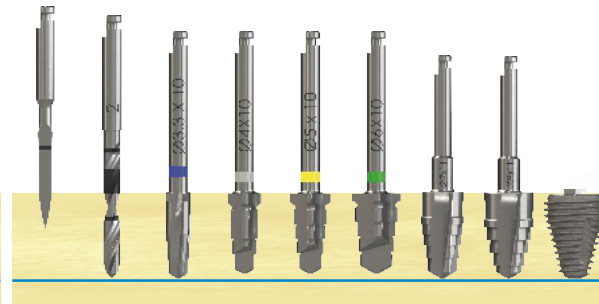

Dense Bone

D-3Spade
Ø2.0mm
Ø3.3mm
Ø4.0mm
Ø5.0mm
Ø6.0mm
Ø7.0mm
Ø8.0mm
D-TAP-MAX8-11

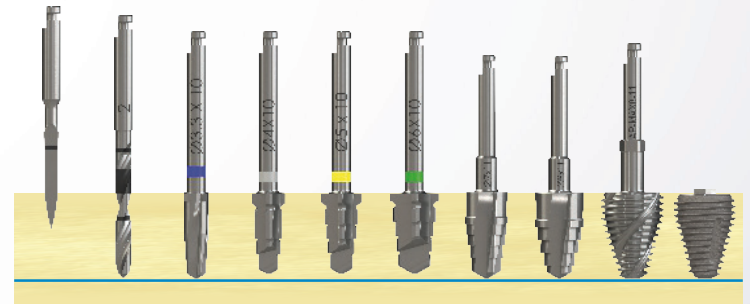

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.
(Illustration is for 13mm implants)