


TRI-NEX 05.0mm Tapered & Parallel Walled Implants - Drill Sequence

Tapered (IA-LH-50)


Soft Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm

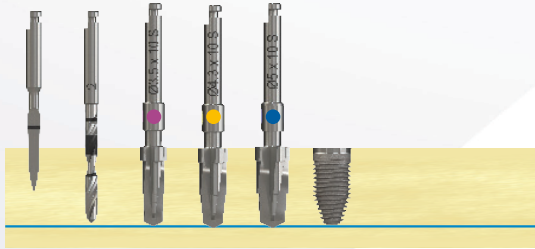


Medium Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm

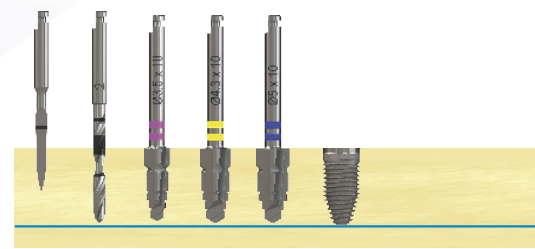

Dense Bone


D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm
D-TAP-L-50

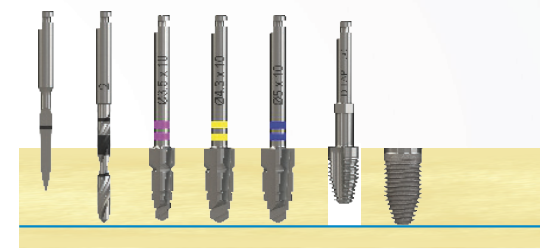

Parallel Walled (IA-LHS-50)


Soft Bone

D-3Spade
Ø2.0mm
Ø3.0mm
Ø4.0mm
Ø4.3mm

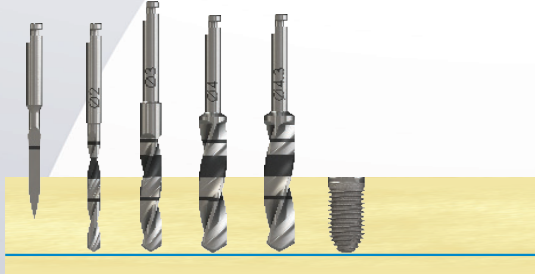


Medium Bone

D-3Spade
Ø2.0mm
Ø3.0mm
Ø4.3mm
Ø4.6mm

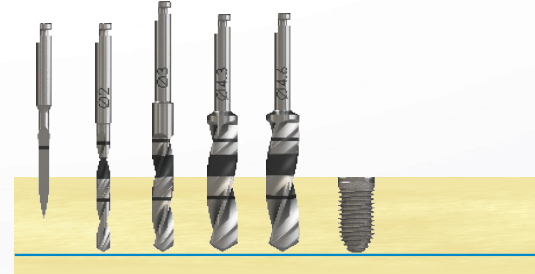

Dense Bone

D-3Spade
Ø2.0mm
Ø3.0mm
Ø4.3mm
Ø4.6mm
D-CSS-5
D-TAP-LS-50

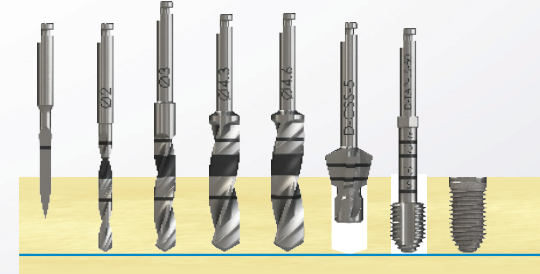

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.
(illustration is for 10mm implants.)