


TRI-NEX Co-Axis® 06.0mm Tapered Implants - Drill Sequence

Tapered (IA50-12d Co-Axis)


Soft Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm

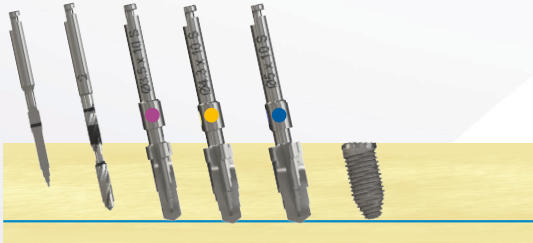


Medium Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm

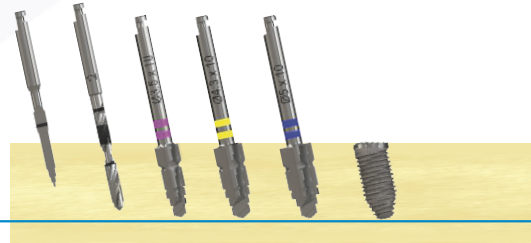

Dense Bone

D-3Spade
Ø2.0mm
Ø3.5mm
Ø4.3mm
Ø5.0mm
D-1AP-L-50

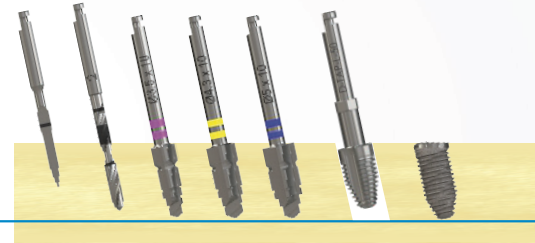

NOTE: Site preparation Sequence recommended by Southern Implants does not replace the judgement and experience of the surgeon.
(illustration is for 10mm implants.)