

# SIDIGITAL Library

## Dental Wings Installation instructions

### Download & Install the Southern Implants Dental Wings library in 3 Simple Steps

#### STEP 1

LOG INTO  
SIDIGITAL PAGE

#### STEP 2

DELETE OLD  
LIBRARIES  
(FOR EXISTING USERS ONLY)

#### STEP 3

INSTALL NEW  
SIDIGITAL LIBRARY

#### STEP 1


##### STEP 1.1

Navigate to the **CAD/CAM - SIDIGITAL** page

Click [here](#) to go to the library registration page

Or

Visit [southernimplants.com](http://southernimplants.com) and navigate to the webpage by hovering over the CAD/CAM - SIDIGITAL tab on the menu bar and click on "CAD/CAM – SIDigital Library Registration & Access".

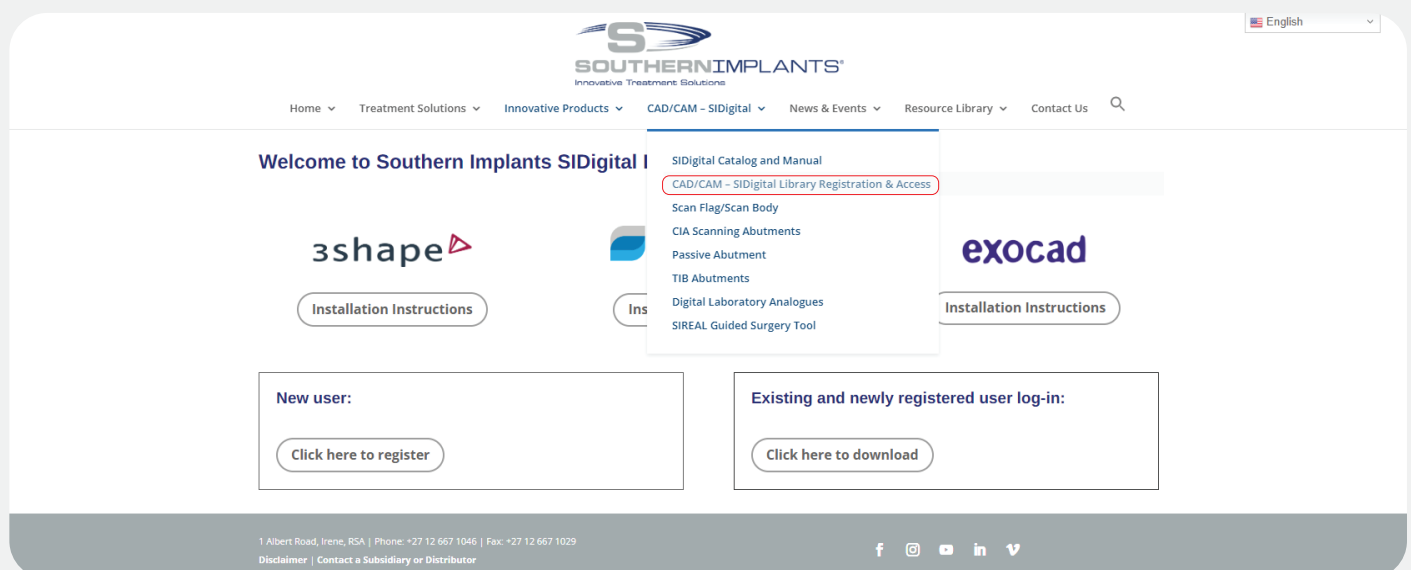


**STEP 1.2**  
**Create new account / log-in**

**FOR NEW USERS**

Create an account by clicking  
"Click here to register".

**FOR EXISTING USERS**

Navigate to the log in screen by clicking  
"Click here to download".

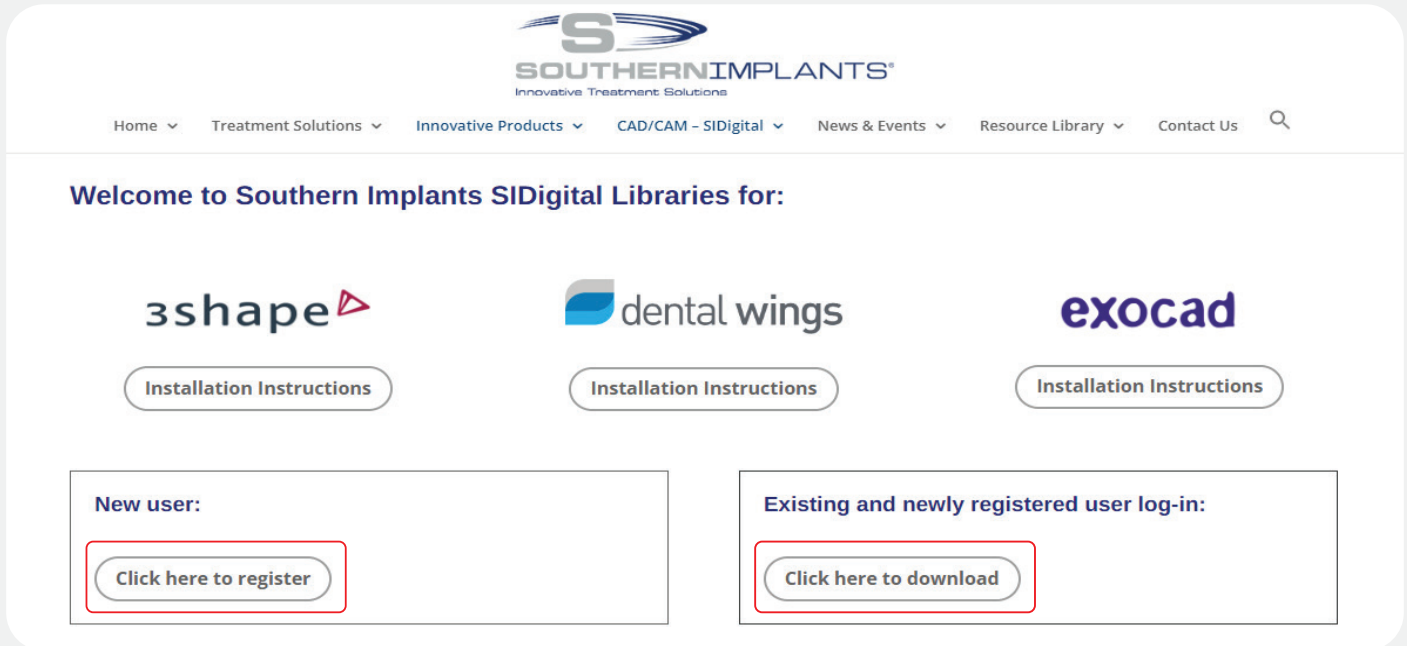

**WELCOME TO SOUTHERN IMPLANTS SIDIGITAL LIBRARIES FOR:**

**3shape** **dental wings** **exocad**

Installation Instructions Installation Instructions Installation Instructions

**New user:**  
[Click here to register](#)

**Existing and newly registered user log-in:**  
[Click here to download](#)

Complete and submit the form. A confirmation email with your login details will follow, check junk folder if not received within 48 hours. Navigate to the login page by clicking the link in the email or by navigating back to  
<https://southernimplants.com/cad-cam-download/>

From: sidigital@southernimplants.com  
Sent:  
To:  
Subject: SI Digital Library Login Details

Good day,

Thank you for registering to SIDigital. To download, click on this link [Access SIDigital](#)

Username:  
Password:

Kind regards,  
SI Digital Team

Enter your password and email address  
(remember to complete the CAPCHA verification)


Log in to download CAD/CAM libraries:

This content is password protected. To view it please enter your password below:

>> [Forgot password?](#)


Email:

Password:

☐ I'm not a robot   
reCAPTCHA  
Privacy - Terms

## STEP 1.3 Download Dental Wings Library



Click on 'Dental Wings Download Latest Version' to initiate the download.


English


Home Treatment Solutions Innovative Products CAD/CAM – SIDigital News & Events Resource Library Contact Us


### Download SIDigital Libraries

Please choose your software: For any queries, contact: [sidigital@southernimplants.com](mailto:sidigital@southernimplants.com)

  
3 Shape *USA Encrypted*  
[Proceed to download](#)  


  
3 Shape  
[Proceed to download](#)

  
Dental Wings  
[Download all](#)

  
Exocad  
[Proceed to download](#)

3 Shape USA Encrypted Change Log

3 Shape Change Log

Dental Wings Change Log

Exocad Change Log

NOTE: A change-log is available of all revision changes to the library at the bottom of the screen.


## STEP 2 DELETING YOUR PREVIOUS LIBRARY

(FOR EXISTING USERS ONLY)

### STEP 2.1

#### Delete the Dental Wings library saved on your local computer

Navigate to your local harddrive (C:\) and delete the existing Southern Implants Dental Wings 'DWOS' folder.

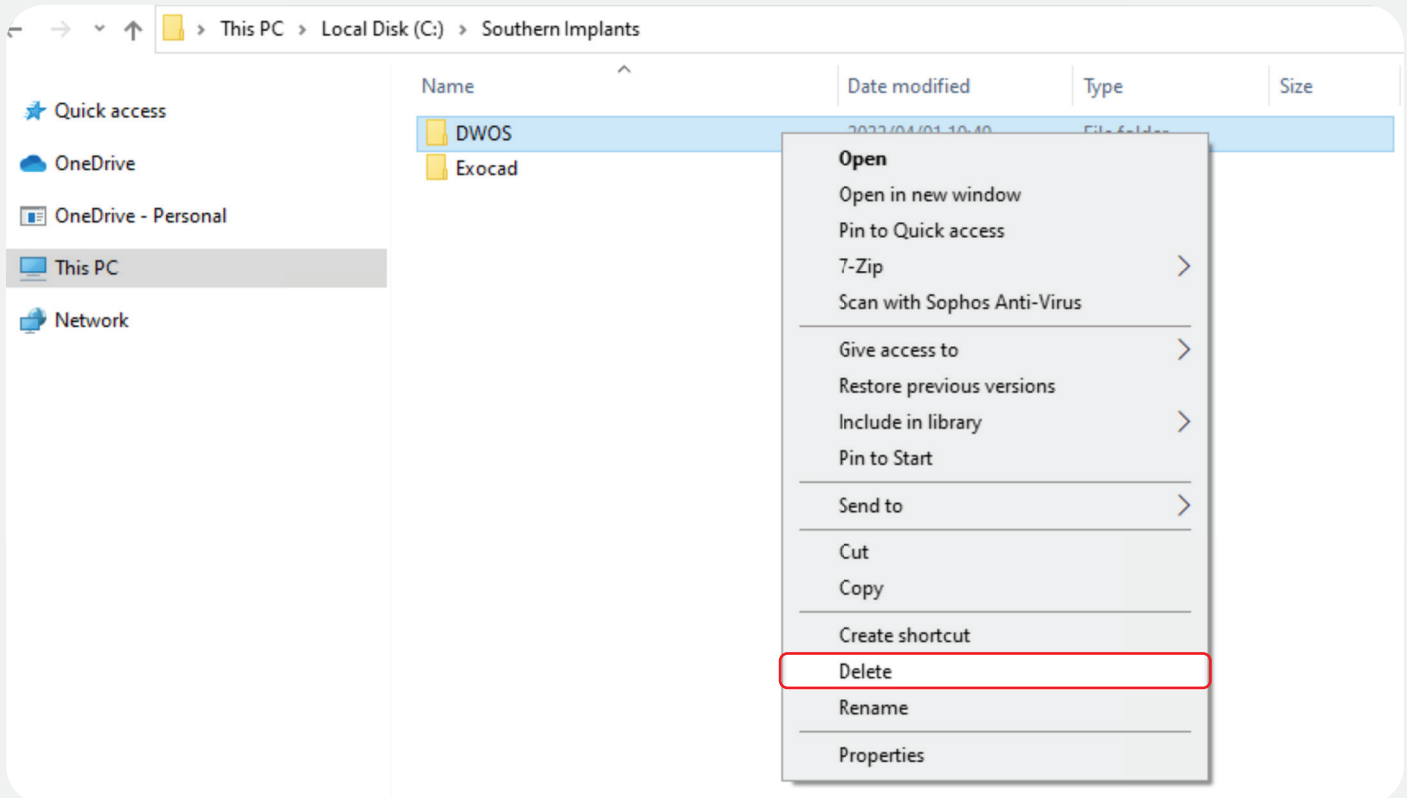


For instances where the previous library was installed in a different location, it is essential that the user navigates to the correct directory and delete the library.

### STEP 2.2

#### Open & navigate to the 'Implant kits' menu


Navigate to the Dental Wings software and within the settings menu, click on 'Implant Kits'.

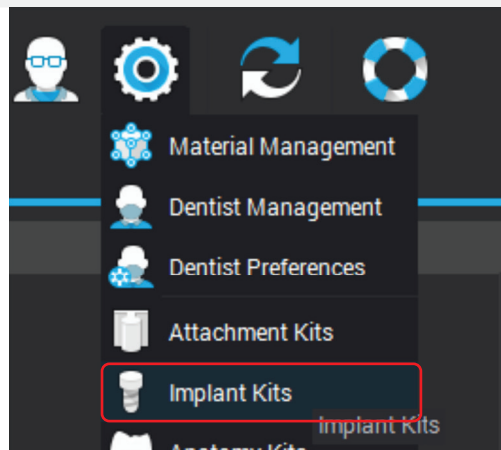

### STEP 2.3

#### Open the dropdown menu


Scroll to the implant kit that you wish to delete and click on the dropdown button.

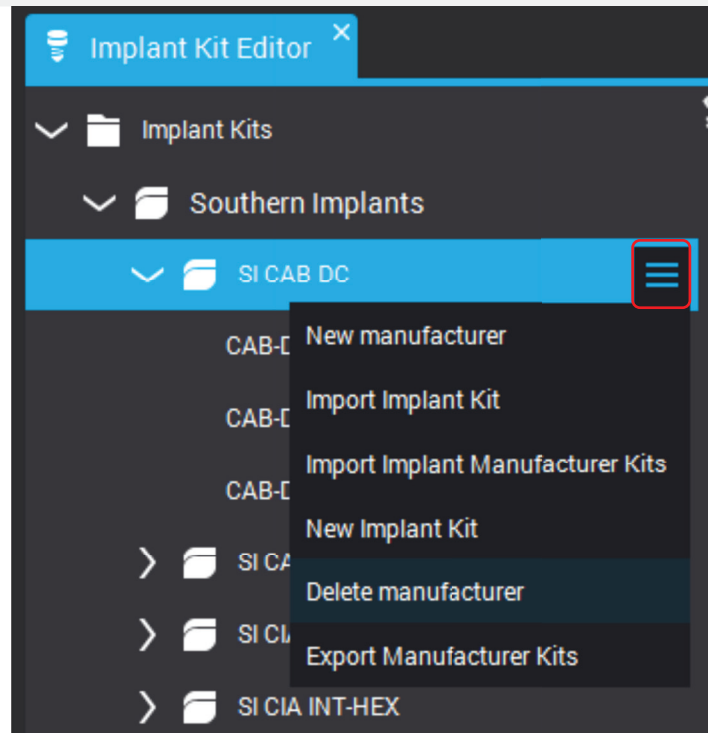

### STEP 2.4

#### Delete the Implant kits

Click on 'Delete manufacturer'.

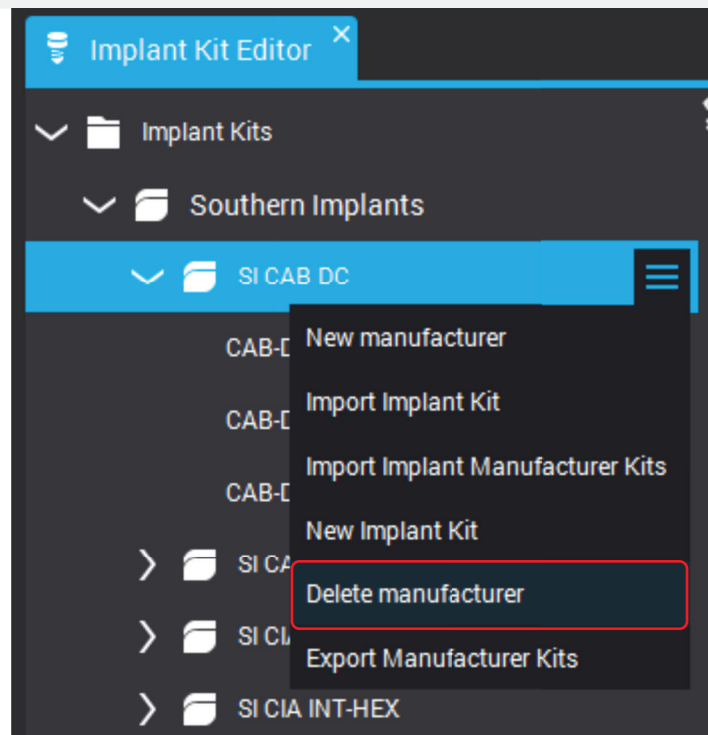

Repeat this step for all necessary implant kits.


**The library is now deleted.**

## STEP 3

### STEP 3.1

#### Navigate to and unzip the Southern Implants library file


Navigate to your downloads folder on your computer and unzip the library file to a folder directory of your choice.

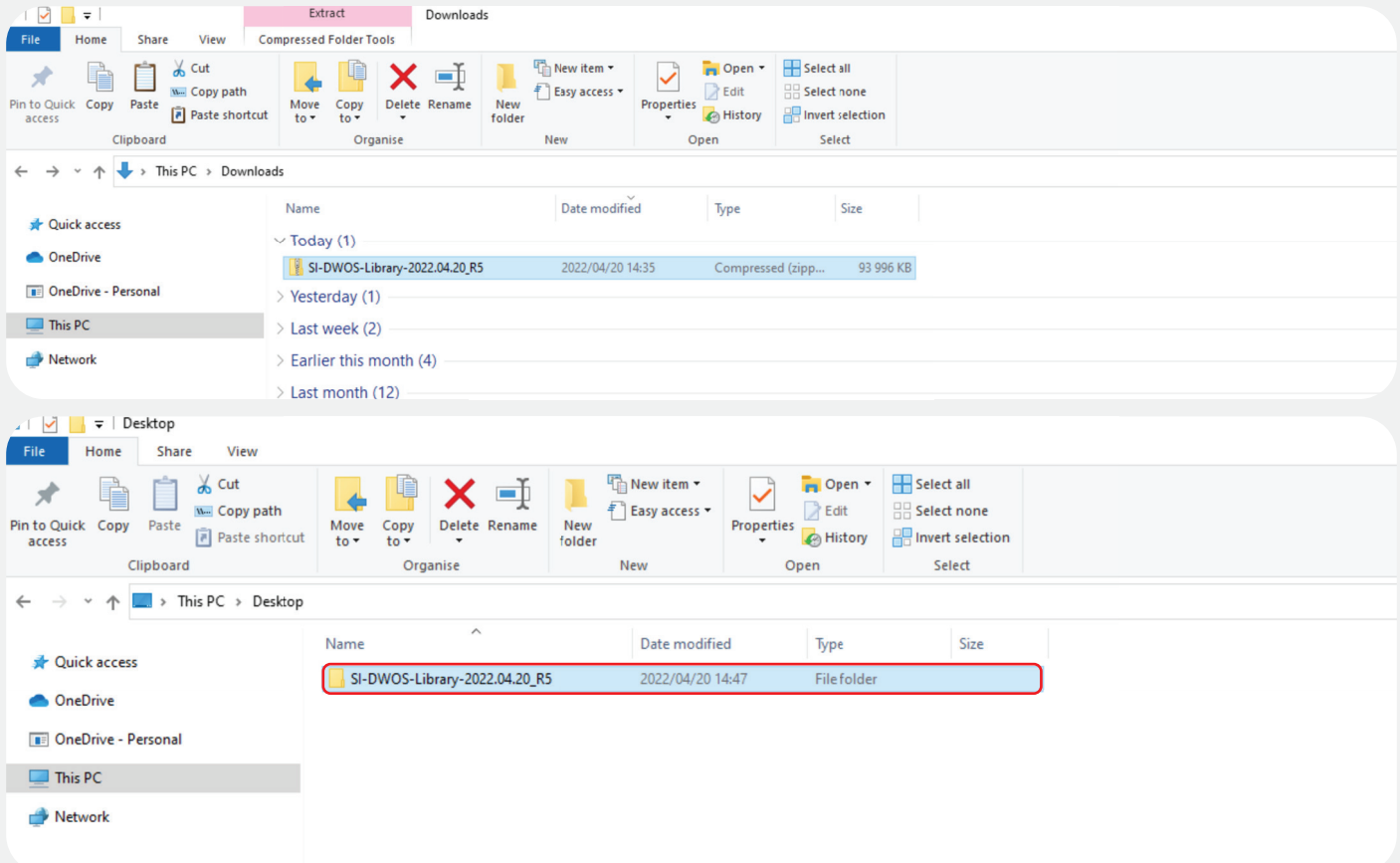

### STEP 3.2

#### Navigate to the Implant kits in the Dental Wings software


Open the Dental Wings software and navigate to 'Implant kits' under the 'Settings' dropdown.

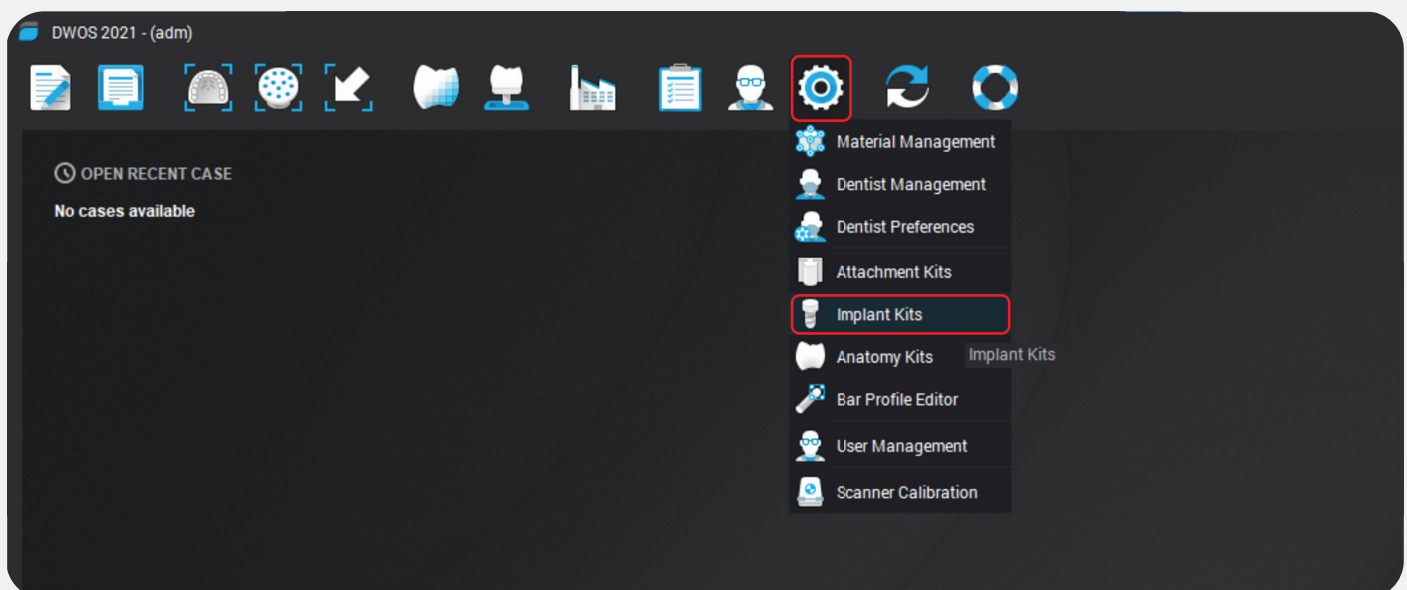


### STEP 3.3

#### Create a new manufacturer

For new installations, a new manufacturer must be created.  
Click the '+' sign at the bottom left of the DWOS application & click on 'New Manufacturer'.

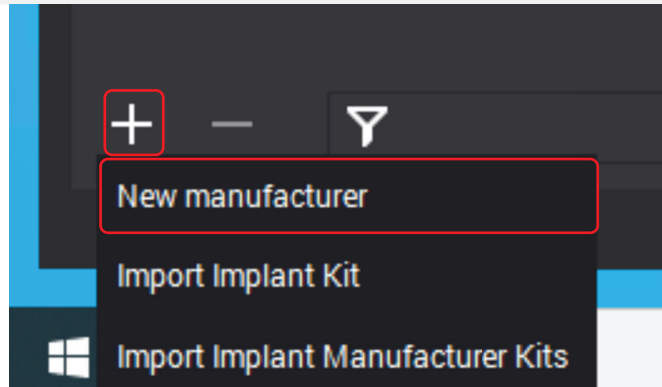


Type 'Southern implants' under manufacturer name and click 'Ok'.

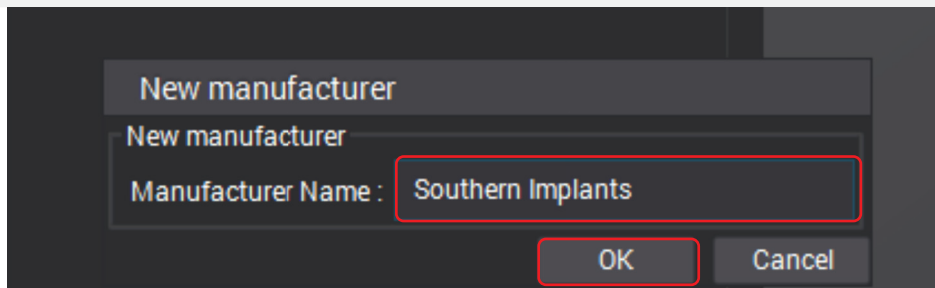


### STEP 3.4

#### Navigate to the 'Import Implant Kit' button

Within the 'Implant Kit Editor' tab screen, select 'Southern Implants'.

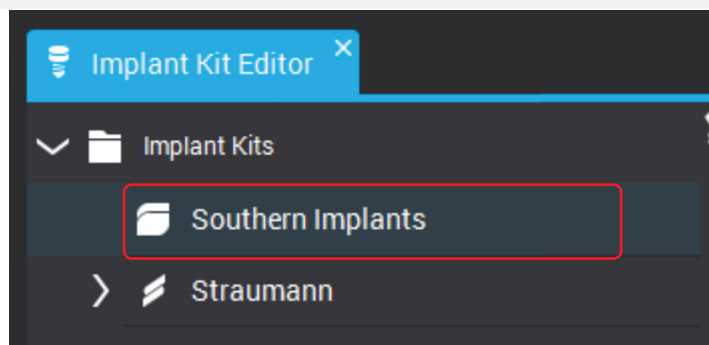


Click 'Import Implant Kit' under the dropdown menu.

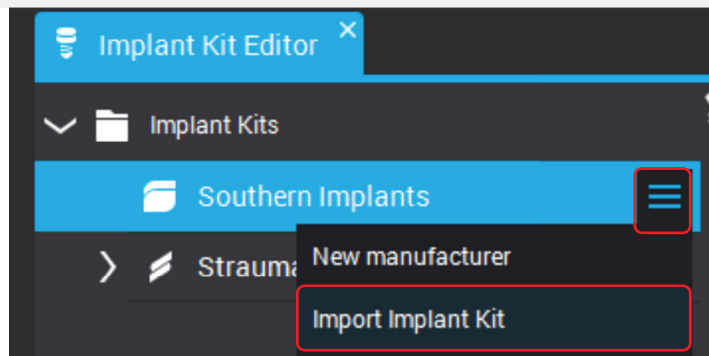


### STEP 3.5

#### Upload the library files into Dental Wings

Navigate to the unzipped SI Dental Wings folder from step 3.1 and open the folder correlating to the abutment you wish to upload into Dental Wings.

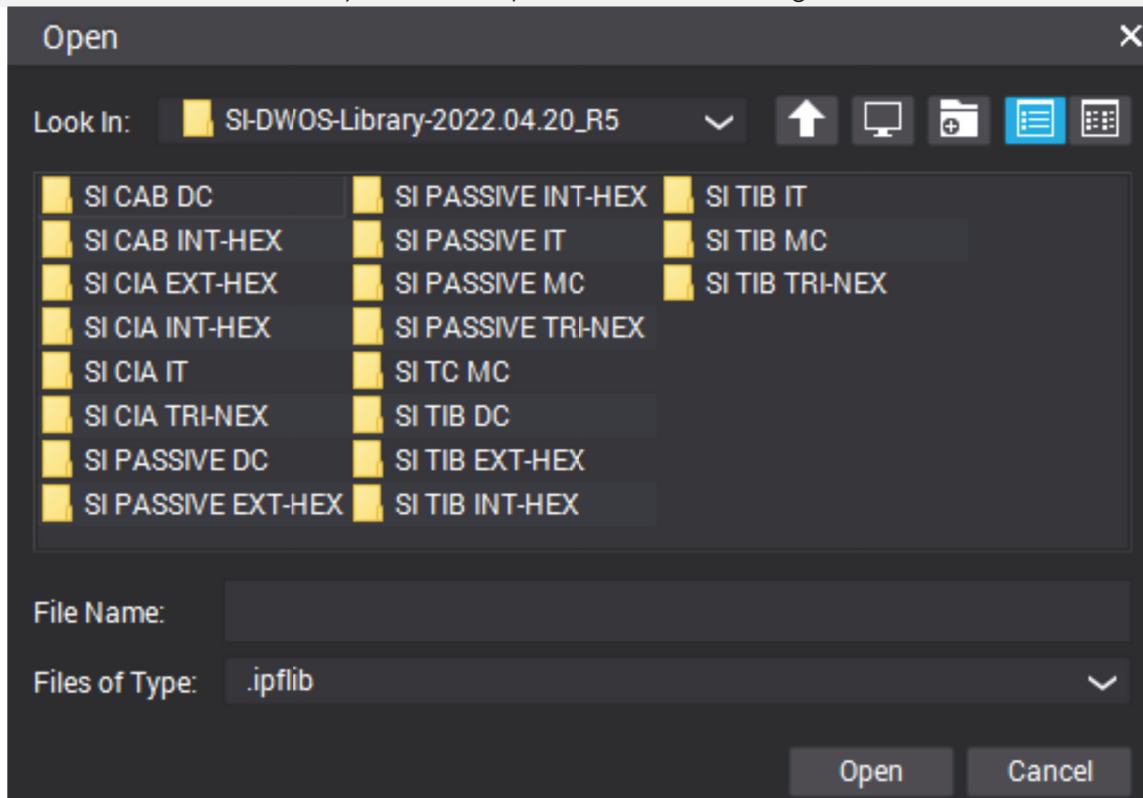

Select all the associated files in the folder and click 'open'.

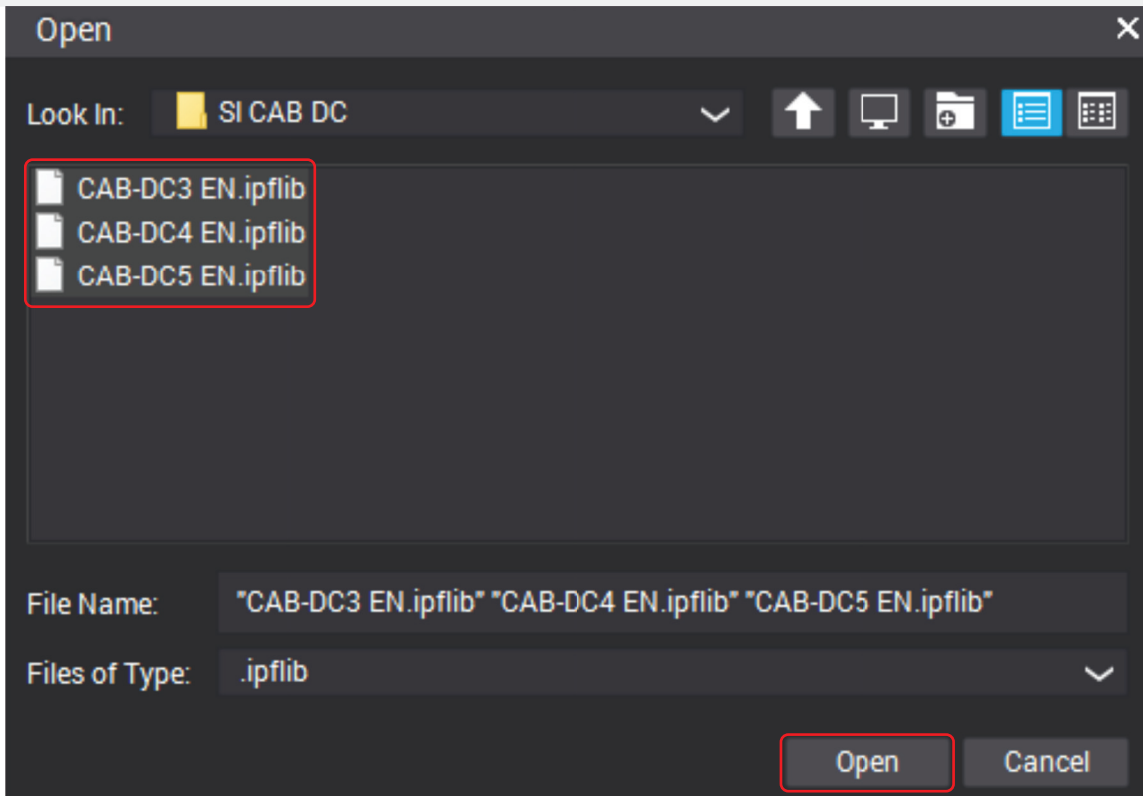


Repeat the upload process for all other abutment files if necessary.



### STEP 3.6

#### Installation confirmation

The updated/installed libraries are listed under 'Southern Implants' within the 'Implant Kit Editor'.

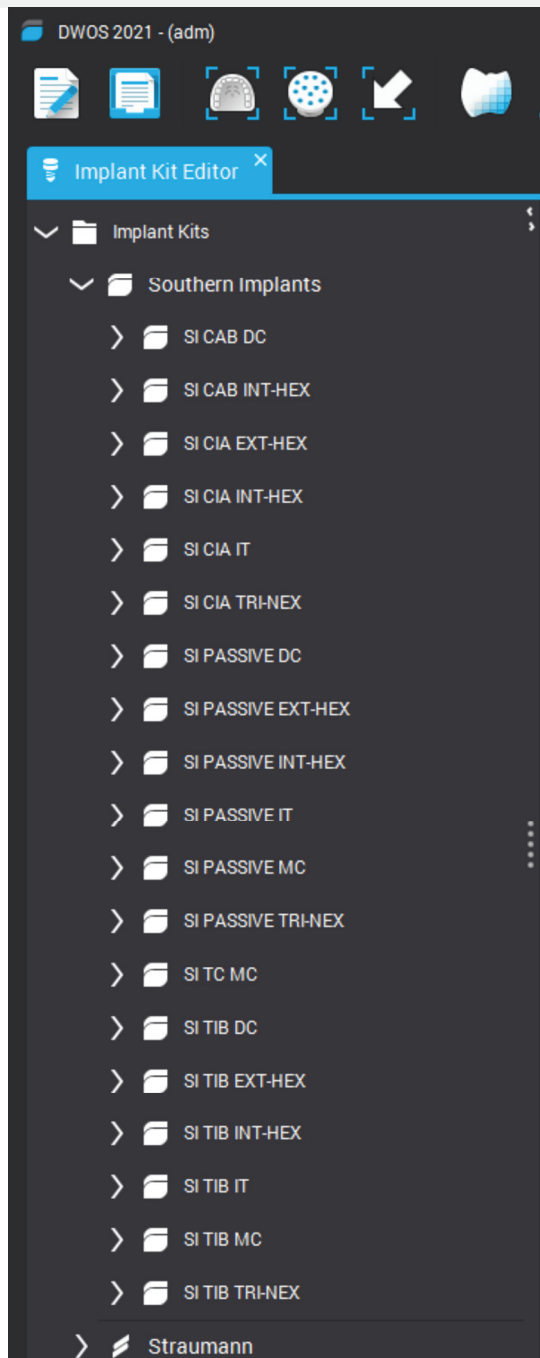

**The Dental Wings library is now successfully installed.**

ADDITIONAL INFORMATION  
SIDigital naming Scheme

Level 1

Main software selection

Company Implant Range  
Southern Implants External Hex

Level 2

Secondary software selection

Company abbreviation Abutment Description (Code Type)  
SI IP Angulated Screw Channel (ASC) "SFT-EX-30"  
Platform Type Scan Flag Type

Level 3

Code explained

ASC-EX-30 (H8)  
ASC-EX-30 (H6)  
ASC-EX-30 (H4.5) ] Modifiable heights