





## **EXTERNAL HEX | IBR24D NAZALUS PROTOCOL**

Site preparation sequence for a 11mm Ø4.0 mm implant

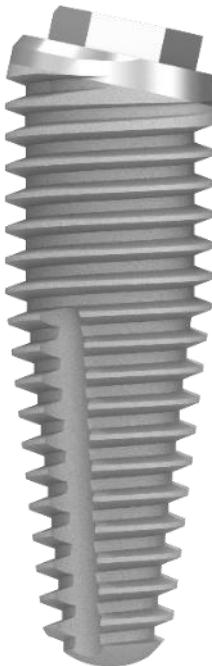

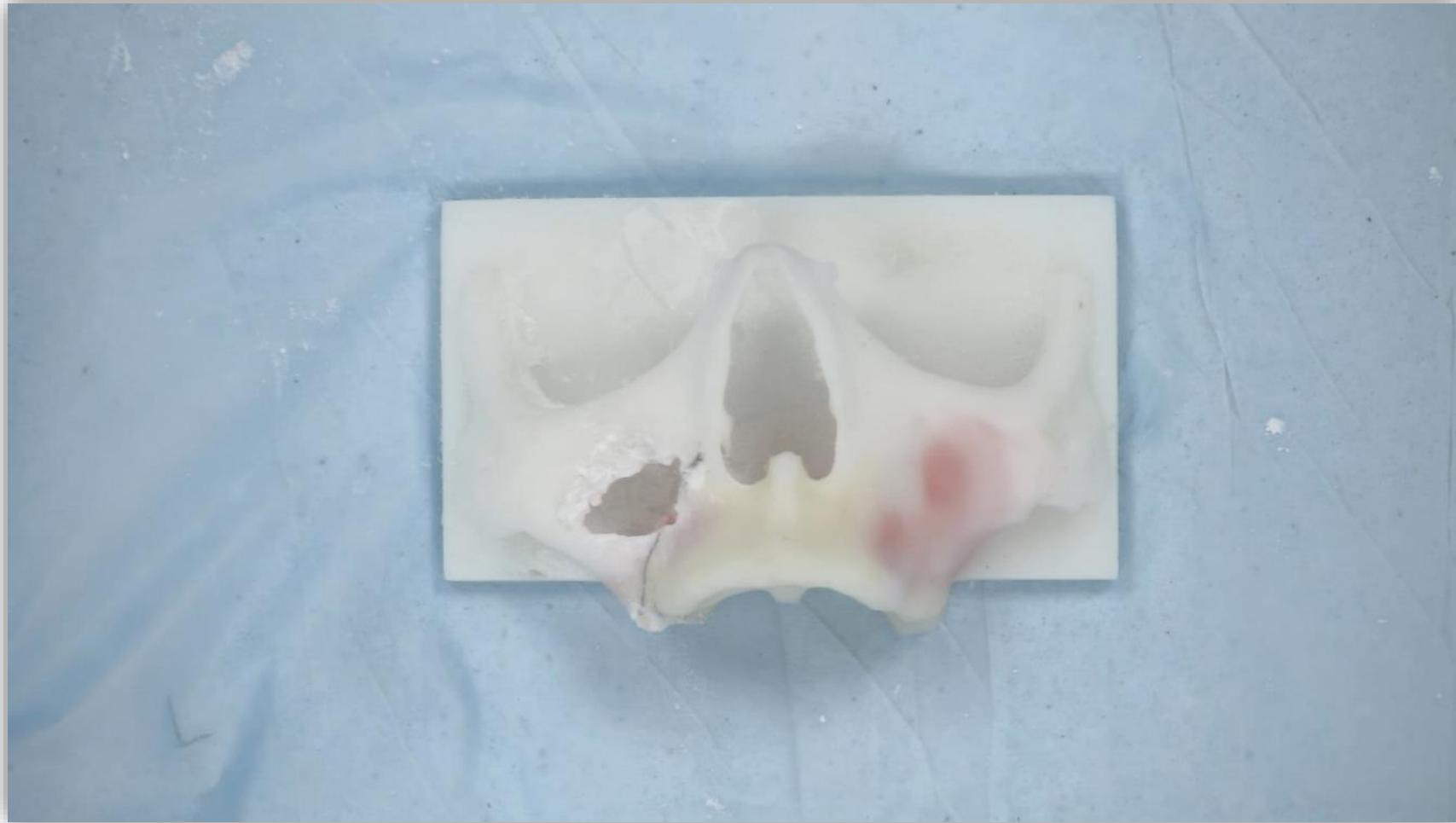


## **STEP 1**

Orientation of the model



**SOUTHERN IMPLANTS®**  
Innovative Treatment Solutions

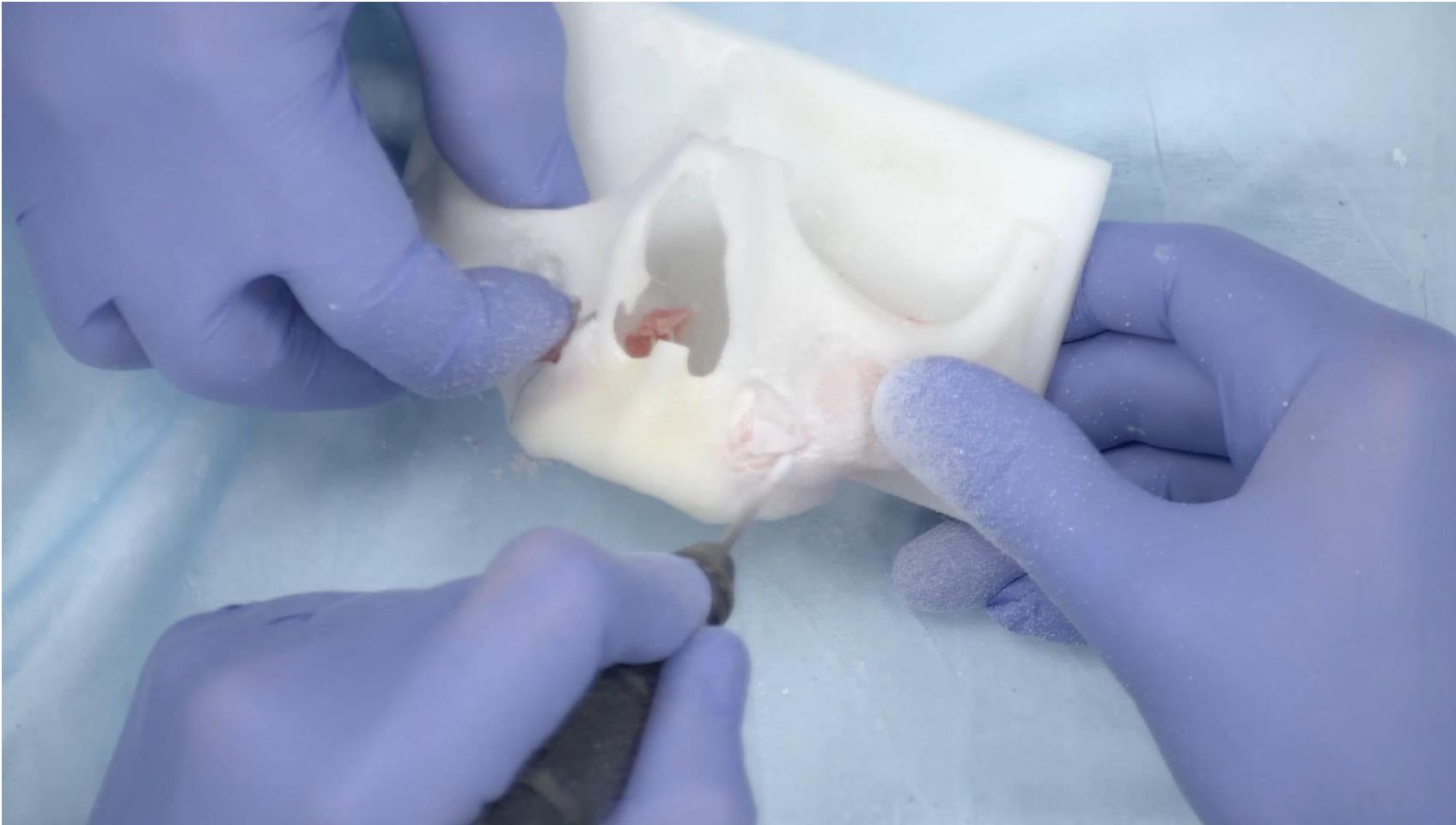


## **STEP 2**

Creating a window

2



**SOUTHERN IMPLANTS®**  
Innovative Treatment Solutions

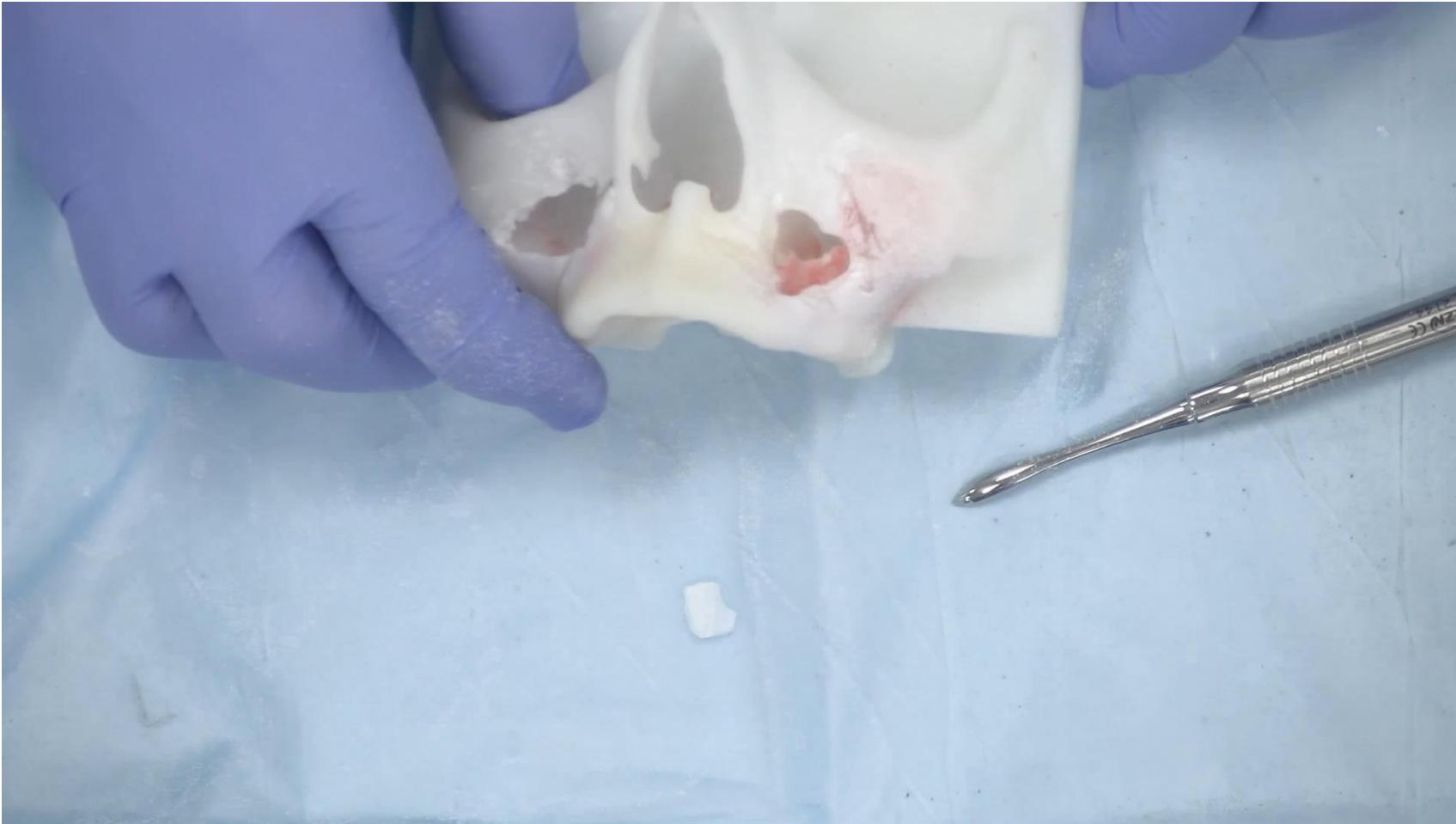


## **STEP 3**

Plan the orientation of the Nazalus implants

3



**SOUTHERN IMPLANTS®**  
Innovative Treatment Solutions

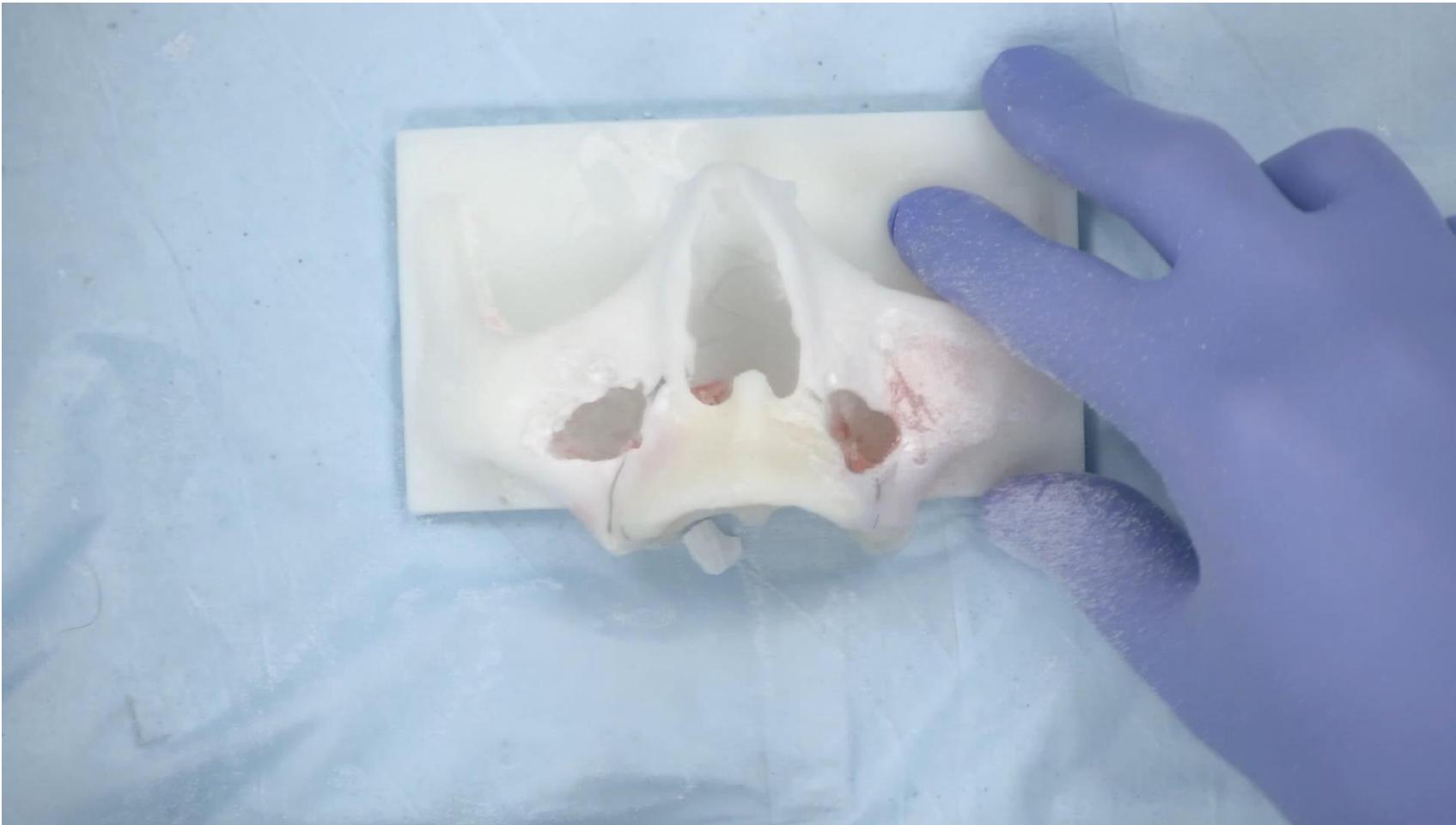

## **STEP 4**

Plan the orientation of the Nazalus  
implants






**SOUTHERN IMPLANTS®**  
Innovative Treatment Solutions



## **STEP 5**

2 mm drill in the planned direction and determine implant length


5

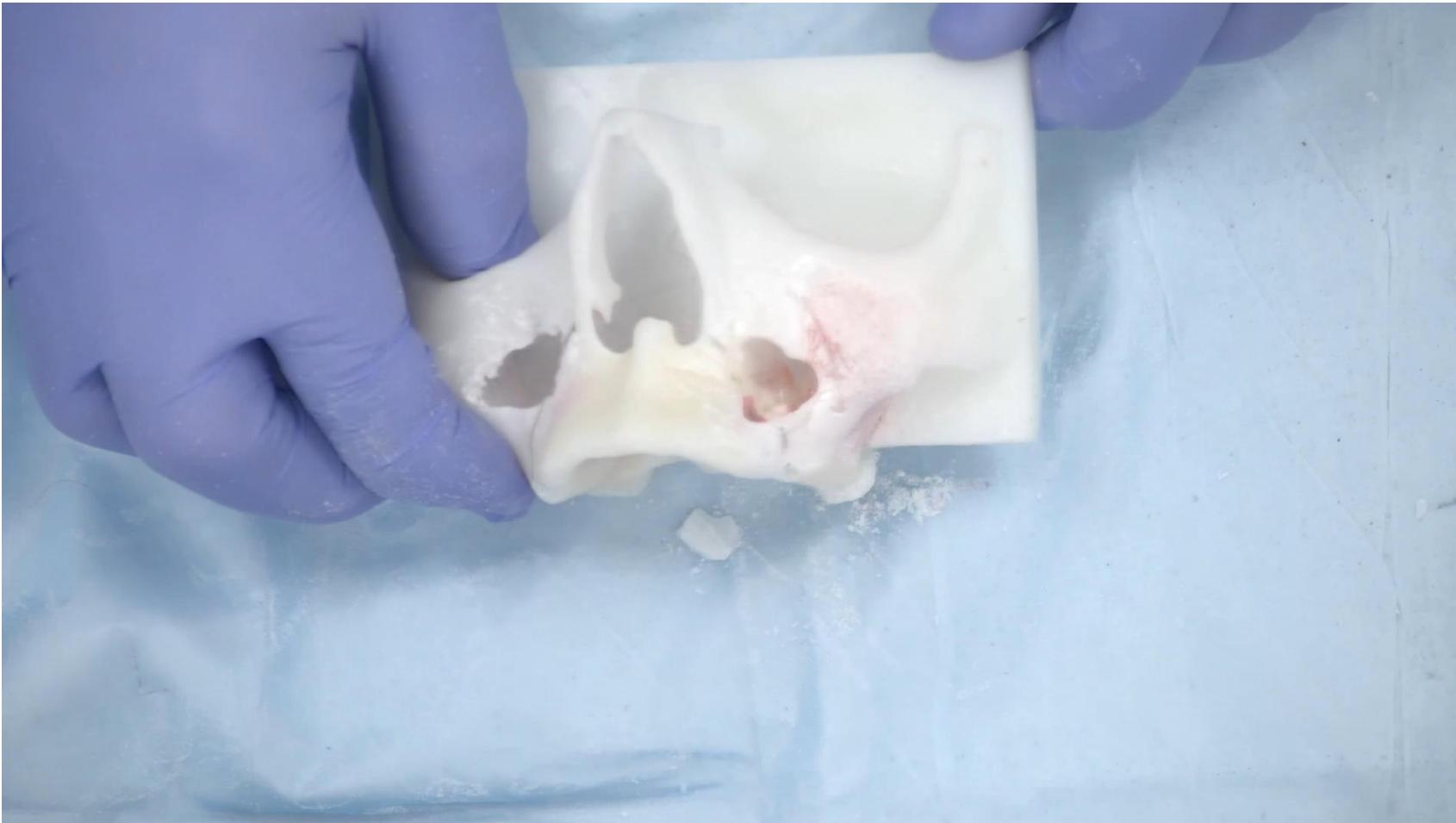

## **STEP 6**

Enlarge the osteotomy with the tapered drills.

6

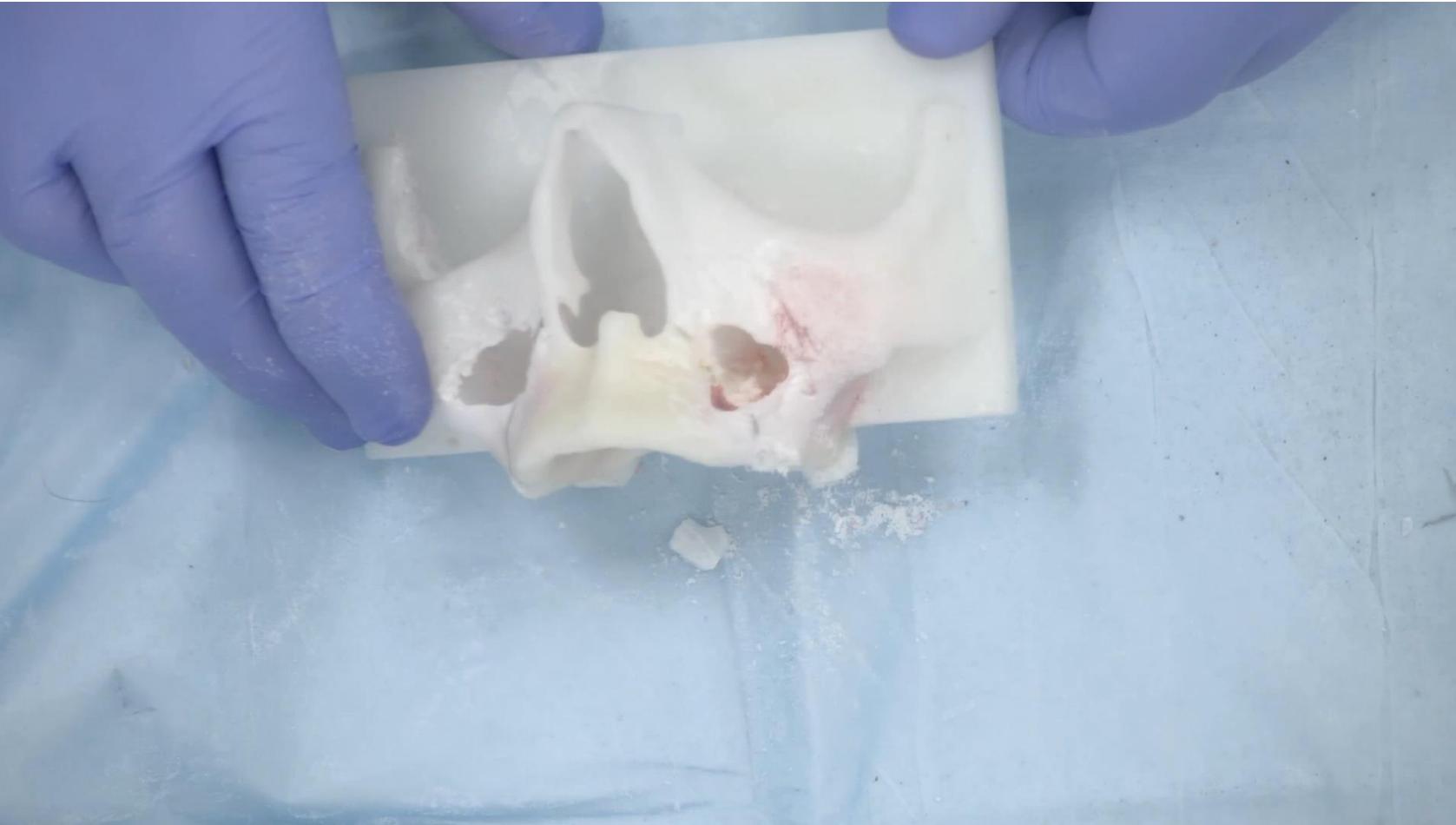


## **STEP 7**

Place the implant





**SOUTHERN IMPLANTS®**  
Innovative Treatment Solutions

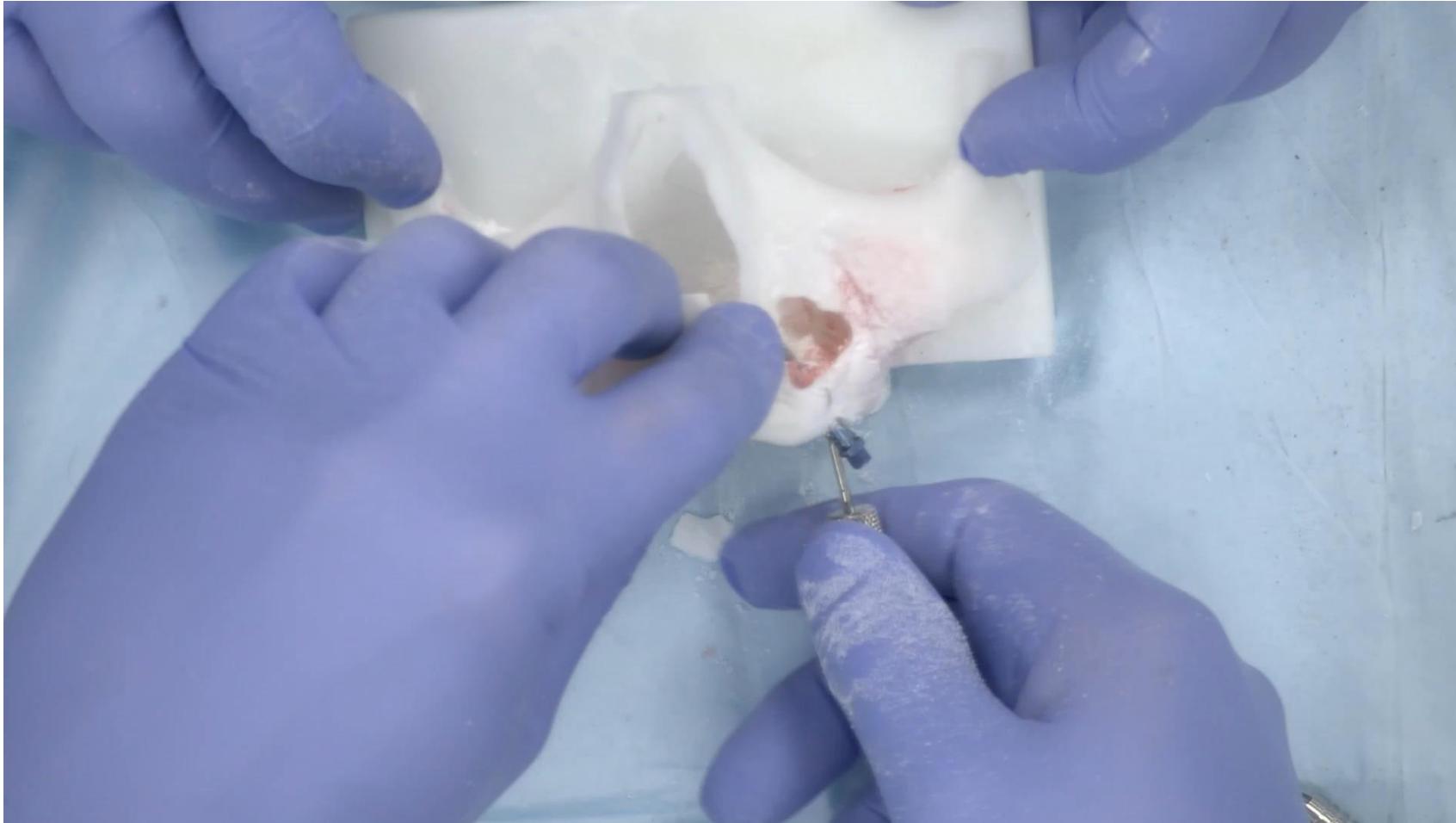

## **STEP 8**

Inspect the implant direction and borders.

8



**SOUTHERN IMPLANTS®**  
Innovative Treatment Solutions



## **STEP 9**

Confirm the direction of the prosthetic angulation.

9